


# ActiLymph®

## Armsleeve prescriber's guide

### Indications for use

**ActiLymph® Armsleeve Class 1 (18-21mmHg)** - Provides light compression and should be used for early/mild lymphoedema, where lymphoedema is light to moderate with little shape distortion. Suitable for lymphoedema, lipoedema, prophylaxis, maintenance therapy, palliative use and patients who are pressure sensitive.

**ActiLymph® Armsleeve Class 2 (23-32mmHg)** - Provides medium compression and should be used in cases of moderate to severe lymphoedema where there may be some shape distortion. **ActiLymph® Class 2 (23-32mmHg)** hosiery can be used as part of the treatment for, and to provide maintenance therapy of lymphoedema.

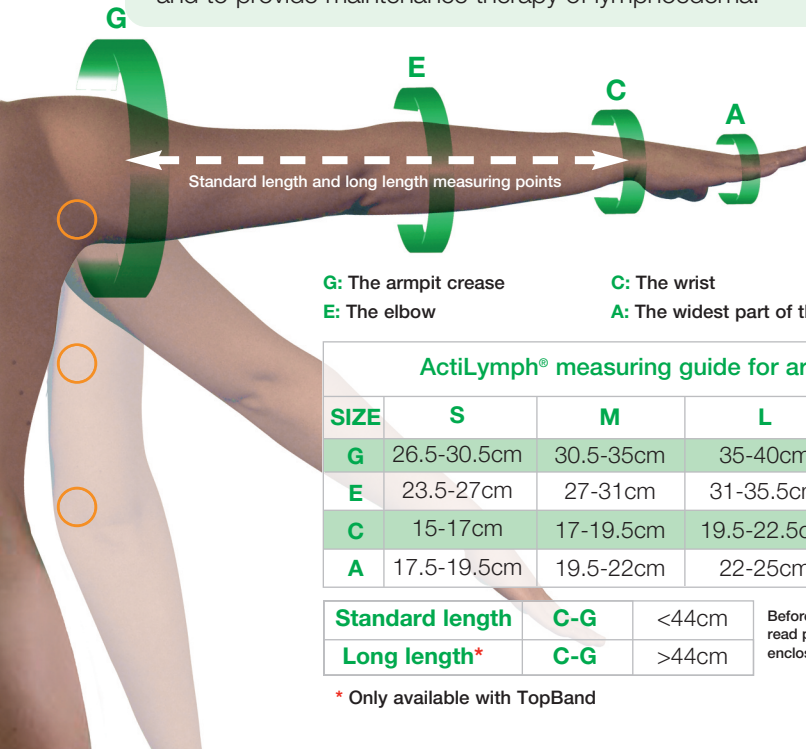

**G:** The armpit crease

**E:** The elbow

**C:** The wrist

**A:** The widest part of the hand

ActiLymph® measuring guide for armsleeves

SIZE	S	M	L	XL
<b>G</b>	26.5-30.5cm	30.5-35cm	35-40cm	40-45cm
<b>E</b>	23.5-27cm	27-31cm	31-35.5cm	35.5-41cm
<b>C</b>	15-17cm	17-19.5cm	19.5-22.5cm	22.5-26cm
<b>A</b>	17.5-19.5cm	19.5-22cm	22-25cm	23-26cm

**Standard length**    **C-G**    <44cm

**Long length\***    **C-G**    >44cm

Before applying armsleeves, please read patient information leaflet enclosed within the box.

\* Only available with TopBand



# ActiLymph®

European Class armsleeve range

## Sizes & PIP Codes

Product Description (Colour - Sand)	Small	Medium	Large	ExLarge
Class 1 Standard (CG)	317-3150	317-3309	317-3317	337-6928
Class 1 Standard (CG) TopBand	317-3325	317-3333	317-3341	336-8917
Class 1 Standard (CG) TopBand Long	342-8794	342-8802	342-8810	342-8828
Class 1 Combined Standard (AG)	317-3358	317-3366	317-3374	-
Class 1 Combined Standard (AG) TopBand	317-3382	317-3390	317-3408	-
Class 1 Combined Standard (AG) TopBand Long	342-8836	342-8844	342-8851	-
Class 2 Standard (CG)	317-3416	317-3424	317-3432	336-9345
Class 2 Standard (CG) TopBand	317-3440	317-3457	317-3465	336-9352
Class 2 Standard (CG) TopBand Long	342-8885	342-8893	342-8901	342-8919
Class 2 Combined Standard (AG)	317-3473	317-3481	317-3499	-
Class 2 Combined Standard (AG) TopBand	317-3507	317-3515	317-3523	-
Class 2 Combined Standard (AG) TopBand Long	342-8927	342-8935	342-8943	-

## Application of ActiLymph® armsleeve using ActiGlide®



**ActiGlide® (PIP code 284-5196)**  
is available on FP10/GP10.

Call our customer care line: **08450 606 707**  
International enquiries: **+44 1283 576 800**  
or visit our website at: **www.Lohmann-Rauscher.co.uk**