

## Cellona® Shoe

Safety and stability.



Adjustable



Non-slip



Washable



Cost effective



Available  
on GP10  
& FP10



The Cellona® Shoe provides stability, protection and comfort for patients with bandaged feet. Easily adjusted to either open-toe or closed-toe wear, the Cellona® Shoe comes in a range of sizes from Child's size 7½ to Men's size 13.



Non-slip



Adjustable



Washable



Cost effective

## Protection and safety

- Firm, reliable base and support
- Non-slip sole even on wet surfaces
- Creates a good foothold, making it easier for patients to walk and encourages their mobility

## Hygienic and hardwearing

- Hand washable at 30°C
- Do not tumble dry
- Made of hardwearing materials
- Will last for 6 months use

## Saving Money

- Offering the best quality at the best price

## Fully adjustable for comfort

- Closed shoe or open sandal
- Protection and comfort against wet and cold weather outside
- Also comfortable in hot weather or when inside
- Full range of sizes
- Fits left or right foot
- Adjustable hook and loop fastening
- Will fit securely to all foot shapes

## Measuring and fitting the Cellona® Shoe

Measure the bandaged foot length and use the following chart to determine which shoe size to order. This will usually be a larger UK shoe size than the patient normally wears.

To prevent tripping, trim the adjustable length of hook and loop ankle fastening tape as required using scissors.

## Cellona® Shoe Sizes

Bandaged foot length	Bandaged foot circumference	Cellona® shoe size	UK shoe size guide	European shoe size guide	REF	PIP Code	NHS Hospital Code
up to 30cm	up to 40cm	XL	9-13	43-47	16476	385-9501	CVL166
up to 28.5cm	up to 40cm	L	7.5-9	41-43	16475	385-9493	CVL167
up to 26.5cm	up to 40cm	M	6-7.5	39-41	16474	385-9485	CVL168
up to 24cm	up to 34cm	S	3-6	36-39	16473	385-9477	CVL169
up to 22.5cm	up to 28cm	XS	11.5-3	30.5-36	16472	385-9451	CVL127
up to 20cm	up to 25cm	Petite	up to 11.5	up to 30.5	16471	385-9469	CVL126

## Closed-toe and open-toe adjustment

The tongue of the shoe can be worn over the bandaged foot for greatest protection of the dressing and toes. Alternatively, the tongue can be laid down into the bottom of the shoe so that the bandaged foot is placed on top of the tongue. This provides the patient with the open-toed sandal option.



Call our customer care line: **08450 606 707**  
 International enquiries: **+44 1283 576 800**  
 or visit our website at: **www.Lohmann-Rauscher.co.uk**

